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Mode d'emploi.....	2
Instructions for use.....	7
Bedienungsanleitung.....	12

# **7T100**

Service commercial :

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Tension de service Maximum : 300 V

## CONTENTS :


- 1- Please read these recommendations before using breath switch 7T100
- 2- The kit includes
- 3- Installation
  - 3-1- of the multipurpose vice
  - 3-2- of the filtering system
- 4- Use
- 5- Requirements for hygiene and maintenance
- 6- Technical characteristics

### 1- PLEASE READ THESE RECOMMENDATIONS BEFORE USING BREATH SWITCH 7T100 :

- It is essential to check everyday that the breath switch 7T100 works properly and that the multipurpose vice and the black hose are firmly fastened.
- The breath switch 7T100 is convenient for a non-continuous use, i.e. the mouthpiece must not always stay in the user's mouth.
- Epileptic persons or persons with sudden and/or uncontrolled head movements must not use the breath switch 7T100.
- Breath switch 7T100 must be used with a nurse call system or electronical technical aids in conformity with the standards in force.
- Breath switch 7T100 must not be used in vehicles, and particularly in wheelchairs, when they are moving.
- Breath switch 7T100 must be used only with parts referenced in the kit and supplied by Protéor.
- Power supply : alternating 24 V maxi – 0,5 A maxi.

### 2- THE KIT INCLUDES



- Rep. 1 : Multipurpose vice (After-sales service : Protéor ref. 7T22701)  
Rep. 2 : Black hose  
Rep. 3 : Antibacterial and antiviral filter (After-sales service : Protéor ref. 7T10002)  
Rep. 4 : Hose (After-sales service : Protéor ref. 7T10003)  
Rep. 5 : Mouthpiece - large size (After-sales service : Protéor ref. 7T10004)  
Rep. 6 : Mouthpiece - small size (After-sales service : Protéor ref. 7T10007)  
Rep. 7 : Box  
Rep. 8 : Jack plug  
Rep. 9 : Wrench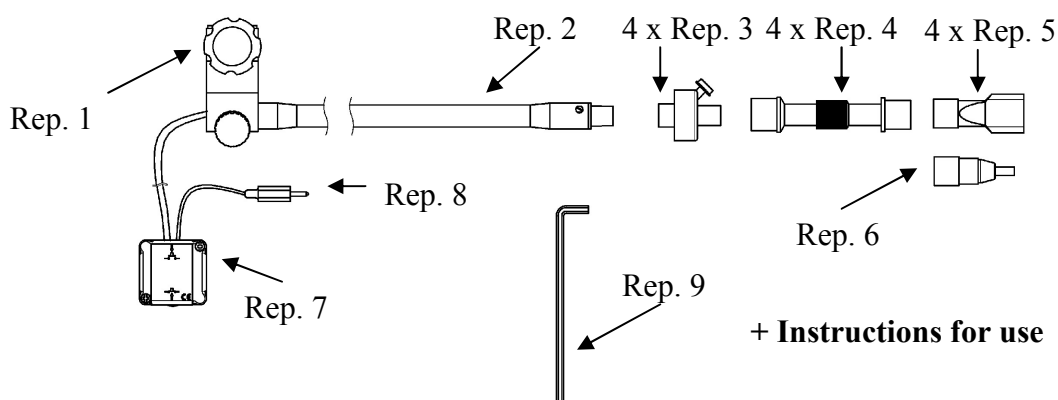



### **3- INSTALLATION :**

#### **3-1 Installation of the multipurpose vice (Rep. 1) :**


The multipurpose vice can be fixed vertically or horizontally on any support (table, table foot, ...) not higher than 40 mm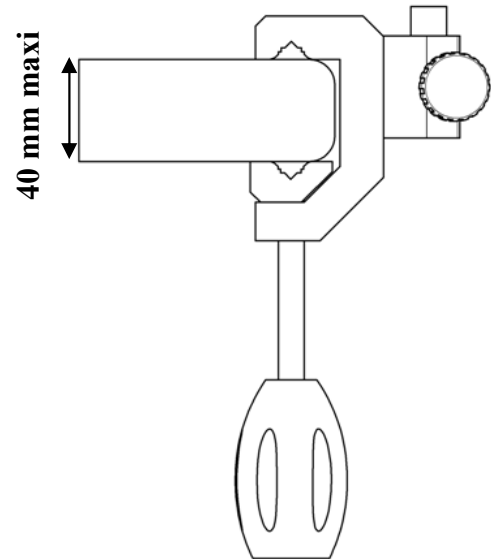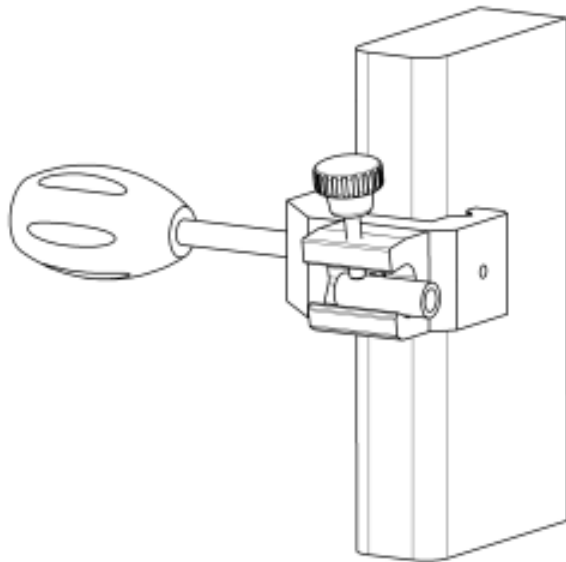


or on tubes with diameters from 12 mm to 35 mm.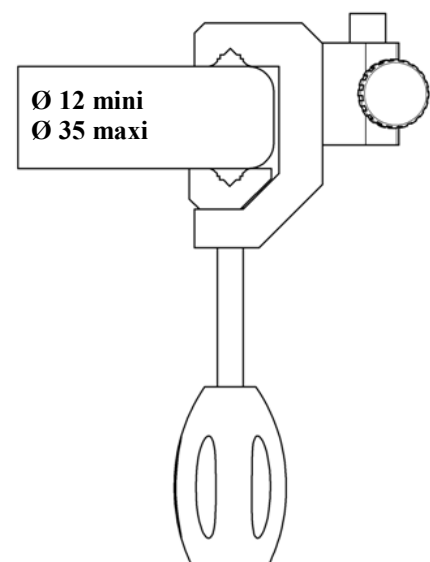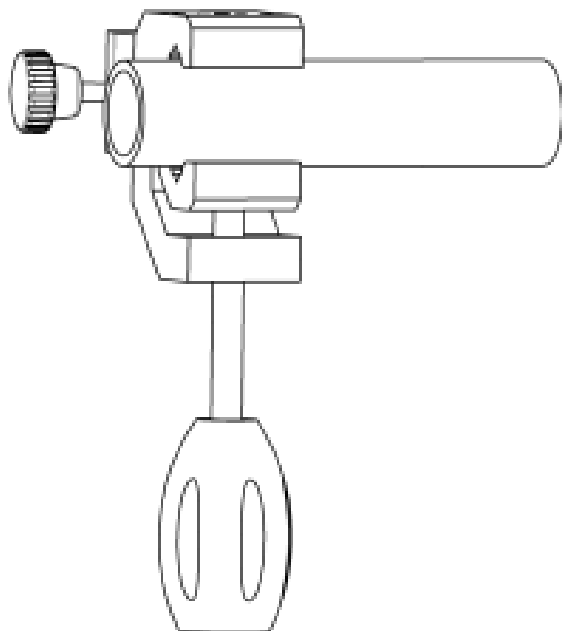

Moreover you can put the tube support vertically or horizontally on the multipurpose vice (using wrench Rep. 9).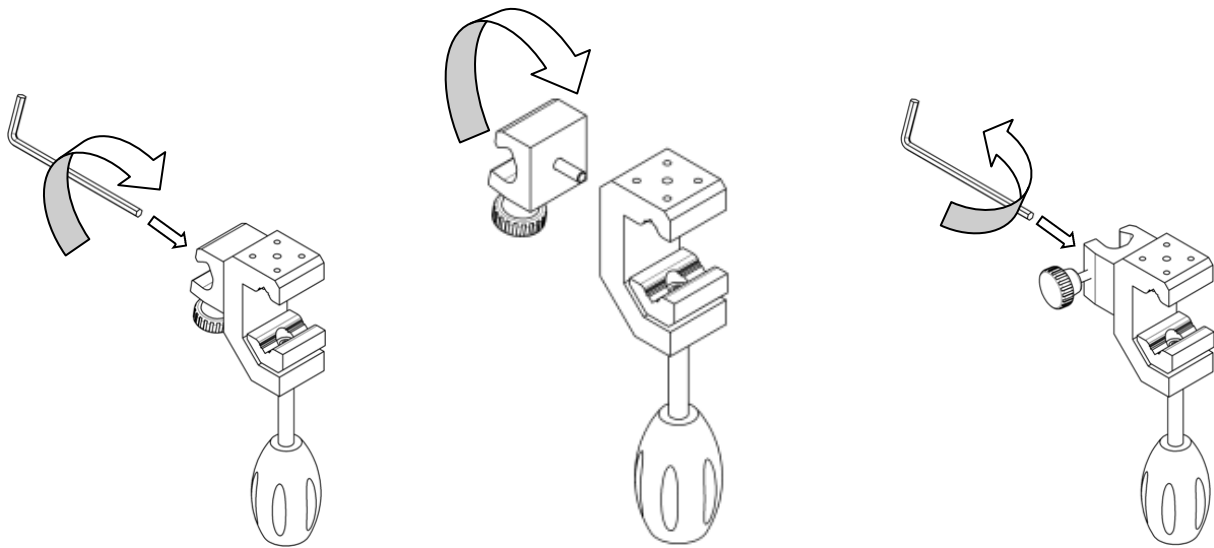

You can also fix the tube holder on the upper face of the multipurpose vice (Rep. 1).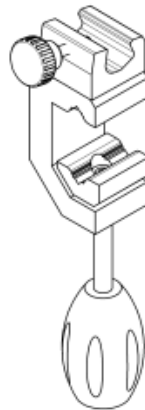


**IMPORTANT :** Always control that the multipurpose vice (Rep. 1) and the black hose (Rep. 2) are firmly fastened.

### 3-2 Installation of the filtering system

**IMPORTANT :** Before mounting the filter, control that the multipurpose vice (Rep. 1) and the black hose (Rep. 2) are firmly fastened.

Hold the black hose (Rep. 2) firmly and proceed as follows without letting it go of : press the filter (Rep. 3) in the grey adapter (see the following diagram), then the hose (Rep. 4) in the filter (Rep. 3), then the large size mouthpiece (Rep. 5) or the small size mouthpiece (Rep. 6) in the hose (Rep. 4)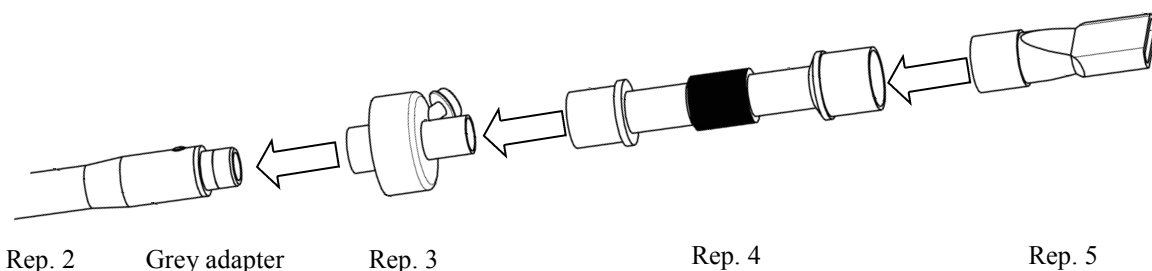

Rep. 2

Grey adapter

Rep. 3


Rep. 4

Rep. 5

#### **4- USE :**

- Place the mouthpiece (Rep. 5 or Rep. 6) near the mouth, so that blowing in it only requires a small movement : Take the multipurpose vice (Rep. 1) in one hand, the end of the black hose (Rep. 2) in the other hand and put the mouthpiece (Rep. 5 or Rep. 6) where you wish.
- You can finalize the adjustment by using the hose (Rep. 4) : Take the filter (Rep. 3) in one hand, the mouthpiece (Rep. 5 or Rep. 6) in the other hand and put it (Rep. 5 or Rep. 6) where you wish.
- When positioning is finalized, connect Jack plug (Rep. 8) to the device you want to control or to the nurse call system.
- Check that the breath switch 7T100 works properly.
- Adjust sensitivity, so that the user's breath be sufficient to make the device work.
- Adjust sensitivity with the adjusting screw on the side of the box (Rep. 7).


- When you turn the adjusting screw clockwise, the switch becomes more sensitive.
- When you turn the adjusting screw counterclockwise, the switch becomes less sensitive.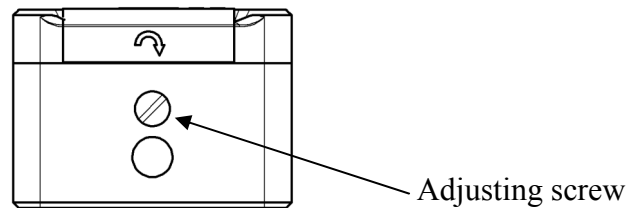


**CAUTION :** The screw must be turned carefully. It is an accurate adjustment and a small screw rotation can modify sensitivity quickly.

**IMPORTANT :** After each adjustment, control that the user's breath makes the device work properly.

- Using the device with a nurse call system where the original calling system cuts the circuit.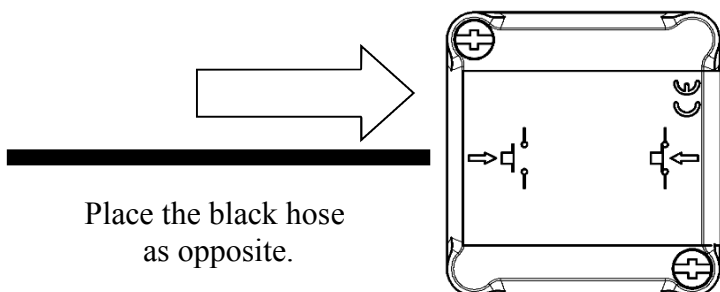

- Using the device with a nurse call system where the original calling system opens the circuit.

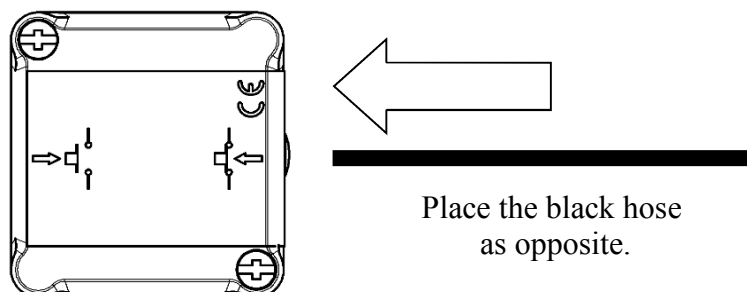**IMPORTANT :** After each adjustment, control that the user's breath makes the device work properly.

## **5- REQUIREMENTS FOR HYGIENE AND MAINTENANCE**

### **Hygiene requirements :**

The filters (Rep. 3), the hoses (Rep. 4), the large size mouthpieces (Rep. 5) and the small size mouthpieces (Rep. 6) must be changed every time that there is another user. If there is only one user, they must be changed at least every 15 days.

The used pieces must be incinerated. Never use them again.

To change these pieces, proceed as indicated in section "installation".

**IMPORTANT :** After the replacement, control that the user's breath makes the device work properly.

### **Maintenance :**

This device requires no maintenance, except changing the filter (Rep. 3), the hose (Rep. 4), the large size mouth-piece (Rep. 5) and the small size mouthpiece (Rep. 6).

Unplug the device before cleaning it and use a dry cloth.

Never use solvent.

In case of abnormal operation, please send the device to us at the indicated address.

Dismantling the device, or trying to dismantle it, would cancel warranty.

**IMPORTANT :** After each cleaning, plug the device again, control that it works properly and that the multipurpose vice and the black hose are firmly fastened.

**List of consumables :** Set of 12 antibacterial and antiviral filters Rep. 3 : Proteor ref. 7T10002=12  
Set of 12 hoses Rep. 4 : Proteor ref. 7T10003=12  
Set of 12 large size mouthpieces Rep. 5 : Proteor ref. 7T10004=12  
Set of 12 small size mouthpieces Rep. 6 : Proteor ref. 7T10007=12

## **6- Technical characteristics :**

When delivered and equipped with its accessories, the breath switch 7T100 complies with the EC directive 1993L0042.

The breath switch 7T100 is expected to be used one million times.

However, if it is used intensively (i.e. more than one hundred times a day), its lifetime will not exceed ten years.

### **Rep. 3 :** Filter 7T10002

Certificates :

G.M.P. RO252/ME

CE 0120 Class 2A

Applied standards :

BS 5750

ISO 5356-1

BS 5724-2.24

ISO/DIS 9360

Efficiency (after 24 hours) :

Virus retention (0.02u) 99.99%

Bacterium retention (0.3u) 99.99%

Without Latex

Storage temperature : from -30°C to +40°C

### **Rep. 4 :** Hose 7T10003

Certificates :

G.M.P. RO252/ME

CE 0120 Class 2A

Applied standards :

BS 5750

ISO 5356

ISO 5367

BS 3849

BS 7143

Without Latex

### **Rep. 5 and Rep. 6 :** Mouthpiece 7T10004 and 7T10007

Certificates :

G.M.P. RO252/ME

CE 0120 Class 2A

Applied standards :

BS 3990

ISO 594

BS 5750

ISO 5356

BS 7143

BS 3849

Without Latex

### **Rep. 7 :** Box

Material : Polycarbonate

Resistance to heat : 140 °C

Pressure switch

Maximum voltage : 250 V

Maximum amperage : 0.5 A

Use temperature : from -5°C to 50°C

Product life : 1 x 10<sup>6</sup> cycles

### **Rep. 8 :** Jack plug

Maximum acceptable voltage : alternating 500 V during 1 mn

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